

ZeroPrep™ Saliva Collection Kit

ZeroPrep™ Saliva Collection Kit is a specimen device intended for the collection of human saliva suitable for use directly in genetic and molecular diagnostic testing. This kit is compatible for use together with the VereRT™ ZeroPrep™ COVID-19 PCR Kit for the diagnosis of COVID-19.



Individually packed Collection Funnel & Collection Tube for convenience and safer way to collect saliva



WE TEST DIRECT!

- Fill to this line**
Spit saliva into the collection funnel
- Pour ZeroPrep™ Saliva Buffer into the Collection Tube
- Cap tightly
- Shake vigorously for 30 sec. Heat at 95°C for 10 min.

VERE RT™ ZERO PREP™ COVID-19 PCR ASSAY

RNA EXTRACTION NOT REQUIRED

Sample Type	Human saliva (oral fluid)
Kit Content	25 units of ZeroPrep™ Saliva Collection Set
Test Compatibility	Pair with VereRT™ ZeroPrep™ COVID-19 PCR Kit
	Time to result within 1.5 hours from start of test
	<ul style="list-style-type: none"> • Able to detect SARS-CoV-2 in less than 40 min cycle time • Uninfected sample can be confirmed by 90 min cycle time • LoD of 10 viral RNA copies per test / 4 viral RNA copies per uL saliva

Product	Catalogue No.	Pack Size	Shelf Life	Storage
ZeroPrep™ Saliva Collection Kit	VRTC-RE025	25 tests per kit	6 Months from Date of Manufacturing	Ambient Condition