



# POINT- OF-CARE

# osom<sup>®</sup>

Strep A Test



## OSOM<sup>®</sup> Strep A Test

### FOR THE QUALITATIVE DETECTION OF GROUP A STREPTOCOCCAL ANTIGEN

#### Features and Benefits

- **Accurate** – High Sensitivity. High Specificity
- **Preventative** – Aids in differential diagnosis of Strep A, preventing the unnecessary use of antibiotics
- **Fast and Simple Workflow** – Easy-to-read two-color results, with results in 5 minutes
- **Convenient** – CLIA-waived with no additional equipment required
- **Exceptional Customer Service** – Online training modules and technical support staffed by certified medical technologists/professionals

#### Performance Characteristics

Sensitivity/Specificity	Sample Type	Limit of Detection	Storage Temperature	Time to Result
96% / 98%*	Throat Swab	<10 colonies**	15-30°C / 59-86°F	5 Minutes

\*Compared to Single Swab Culture

\*\*Refer to IFU



**SENSITIVITY**



**SPECIFICITY**




# OSOM® Strep A Test




Streppy

## SIMPLE TO USE / RESULTS IN MINUTES

**1** JUST BEFORE TESTING, ADD 3 DROPS REAGENT 1 (PINK) AND 3 DROPS REAGENT 2 TO THE TEST TUBE (THE SOLUTION SHOULD TURN LIGHT YELLOW)

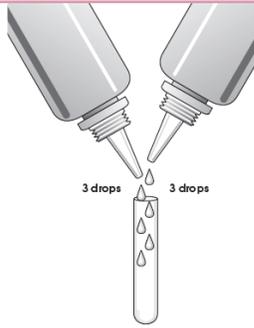


**2** INSERT SWAB AND MIX TEN (10) TIMES. LET STAND FOR 1 MINUTE

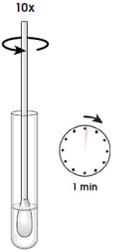

**3** SQUEEZE THE SIDE OF THE TUBE AS SWAB IS REMOVED FROM TUBE



**4** PLACE TEST STICK INTO EXTRACTED SAMPLE INSERT AND READ RESULTS AT 5 MINUTES

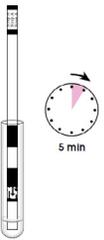

### INTERPRETATION OF RESULTS


#### POSITIVE



#### NEGATIVE



#### INVALID

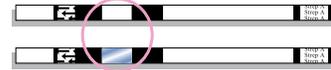

### Ordering Information

Description	Configuration	Catalog Number	CPT Code (US only)
OSOM STREP A TEST KIT	50 tests per Kit	141	87880QW
OSOM STREP A TEST KIT (CE MARKED)	50 tests per Kit	141E	N/A
OSOM STREP A TEST KIT (CE MARKED)	20 tests per Kit	141E-20	N/A

To learn more about all of our OSOM Tests scan the code or go to [osomtests.com](http://osomtests.com)



#### THE AMERICAS

SEKISUI Diagnostics, LLC  
One Wall Street  
Burlington, MA 01803  
Phone: 800 332 1042  
Fax: 800 762 6311

Email: [questions@sekisui-dx.com](mailto:questions@sekisui-dx.com)

#### INTERNATIONAL

SEKISUI Diagnostics (UK) Limited  
Liphook Way, Allington  
Maidstone, Kent, ME16 0LQ, UK  
Phone: +44 1622 607800  
Fax: +44 1622 607801

Email: [info@sekisui-dx.com](mailto:info@sekisui-dx.com)



**SEKISUI**  
DIAGNOSTICS

Because every result matters™

[sekisuidiagnostics.com](http://sekisuidiagnostics.com)

80-8759-00-00 12/21