



POINT- OF-CARE



OSOM® Trichomonas Rapid Test

FOR THE DETECTION OF TRICHOMONAS VAGINALIS ANTIGENS

Features and Benefits

- **Accurate** – High Sensitivity & High Specificity
- **Fast and Simple Workflow** – One-minute hands-on time, with results in 10 minutes
- **Detects the Antigen** – Removes uncertainty associated with live organism
- **Convenient** – Patients have the ability to self-swab to collect sample*
- **Simple** – The only CLIA Waived test for Trichomonas on the market and no equipment needed
- **Exceptional Customer Service** – Online training modules and technical support staffed by certified medical technologists/professionals

Performance Characteristics

Sensitivity/Specificity	Sample Type	Limit of Detection	Storage Temperature	Time to Result
96%/95%**	Vaginal swab	2500 organisms per mL***	15-30°C/59-86°F	10 minutes

*Self-swab claim is only applicable to the CE Marked version.

**Sensitivity/Specificity compared to wet mount

***2500 is a concentration lower than that expected in the vaginal discharge of most positive patients



SENSITIVITY

96%

SPECIFICITY

95%

OSOM® Trichomonas Rapid Test

SIMPLE TO USE / RESULTS IN MINUTES

1 ADD SAMPLE BUFFER TO TEST TUBE

2 INSERT SWAB AND MIX
Let Stand 1 min.

3 REMOVE SWAB FROM TEST TUBE

4 INSERT TEST STICK INTO EXTRACTED SAMPLE
Read Results at 10 min.

POSITIVE

NEGATIVE

Ordering Information

	Configuration	Catalog Number	Storage	CPT Code
OSOM TRICHOMONAS TEST (CE MARKED)	25 tests per kit	181E	Room Temperature	N/A
OSOM TRICHOMONAS TEST	25 tests per kit	181	Room Temperature	87808QW
TRICHOMONAS POSITIVE CONTROL KIT	10 Positive Swabs	182	Room Temperature	N/A

To learn more about all of our OSOM Tests scan the code or go to osomtests.com



THE AMERICAS

SEKISUI Diagnostics,
One Wall Street,
Burlington, MA 01803
Phone: 800 332 1042
Fax: 800 762 6311

Email: questions@sekisui-dx.com

INTERNATIONAL

SEKISUI Diagnostics (UK) Limited,
Liphook Way, Allington
Maidstone, Kent, ME16 0LQ, UK
Phone: +44 1622 607800
Fax: +44 1622 607801

Email: info@sekisui-dx.com



SEKISUI
DIAGNOSTICS

Because every result matters™

sekisuidiagnostics.com